

De OneTouch Reveal[®] mobiele app instellen met Android


ONETOUCH[®]


Stap 1: De mobiele app downloaden

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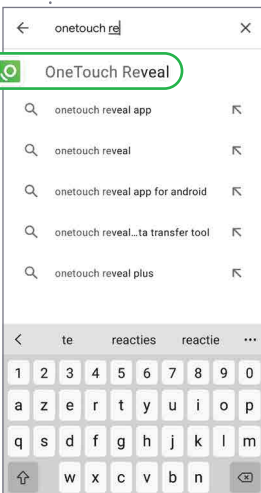B  The screenshot shows the search results for 'onetouch reveal' in the Google Play Store. The search bar at the top contains 'onetouch reveal'. Below the search bar, a list of suggestions is displayed, with 'OneTouch Reveal' highlighted in green. A keyboard is visible at the bottom of the screen.

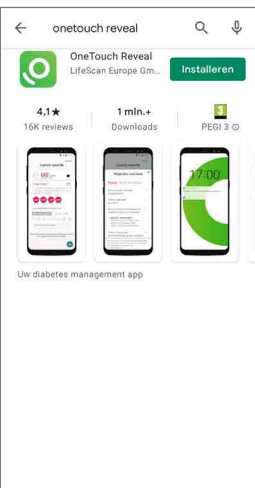C  The screenshot shows the app page for 'OneTouch Reveal' by LifeScan Europe GmbH. The app has a 4.1 star rating from 16K reviews and over 1 million downloads. It is rated PEGI 3. The 'Installeren' (Install) button is visible in the top right corner. Below the app icon, there are three preview images showing the app's interface on a smartphone.

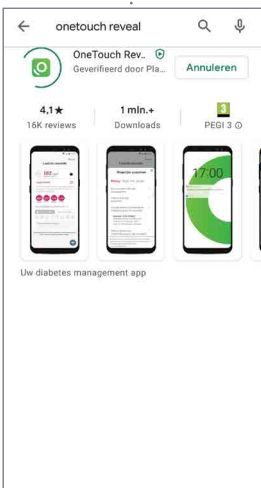D  The screenshot shows the app page for 'OneTouch Reveal' by LifeScan Europe GmbH. The app has a 4.1 star rating from 16K reviews and over 1 million downloads. It is rated PEGI 3. The 'Verwijderen' (Remove) button is visible in the top right corner. Below the app icon, there are three preview images showing the app's interface on a smartphone.

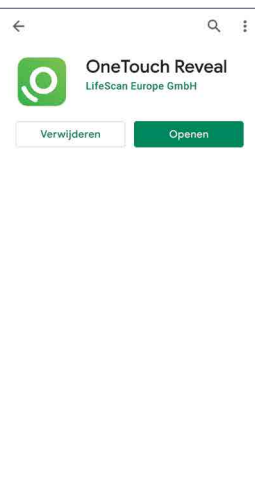E  The screenshot shows the app page for 'OneTouch Reveal' by LifeScan Europe GmbH. The app has a 4.1 star rating from 16K reviews and over 1 million downloads. It is rated PEGI 3. The 'Openen' (Open) button is visible in the top right corner. Below the app icon, there are three preview images showing the app's interface on a smartphone.

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© 2020 LifeScan Belgium BV, BE-DMV-2000013
Het Bluetooth® woordmerk en logo zijn geregistreerde handelsmerken van Bluetooth SIG, Inc., en elk gebruik van deze merken door LifeScan Schotland is onder licentie. Andere handelsmerken en handelsnamen zijn van hun respectieve eigenaars. Google Play en Google Play logo zijn trademarks van Google LLC